

easeScan

For Snow 10 Pro

User's Guide
V1.0



ZOOMAX Technology Co., Limited
9F, Building D, Paradise Software Park,
No.3 Xidoumen Road, Xihu District, Hangzhou, China
310012

Copyright© 2018 ZOOMAX Technology Co., Limited.
All Rights Reserved. ZOOMAX[®] is a trademark of
ZOOMAX Technology Co., Limited in the United States
and other countries. Information in this document is
subject to change without notice.

No part of this publication may be reproduced or transmitted in any forms or by any means electronic or mechanical, for any purpose, without the express written permission of ZOOMAX.

To contact by e-mail, send questions to sales@zoomax.com. To search the knowledge base, please visit www.zoomax.com and select the relevant links.

CONTENTS

1. INTRODUCTION	1
2. ACCESSORIES	3
3. DIRECTION	5
4. SPECIFICATIONS	11

1. Introduction

➤ About your easeScan

Congratulations on purchasing your easeScan, an assistant document scanner designed for Snow 10 Pro. Please read this user's guide carefully before using. This guide will help you understand the easeScan's features better and allow you to get the full benefit of the easeScan and your Snow 10 Pro video magnifier.

➤ About this User's guide

We constantly seek to improve our products and their performance. Therefore, this guide may not be the latest version. You can still download the latest user's guide in the Support section at www.zoomax.com.


If you have any questions after reading this guide, please contact your supplier for further assistance.

2. Accessories

You would find the following items inside the easeScan package box:



easeScan

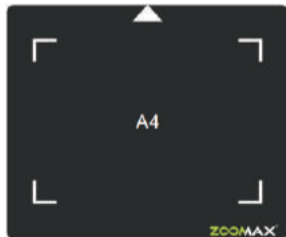

Document mat



USB cable




3. Direction

(1) How to connect easeScan to the Snow 10 Pro

First, turn on your Snow 10 Pro.

Second, use the included USB cable to connect the easeScan with Snow 10 Pro.

- Plug the USB 2.0 (the square end) to the USB port of the easeScan.
- Plug the USB type-c (the small end) to the USB type-c port of the Snow 10 Pro.

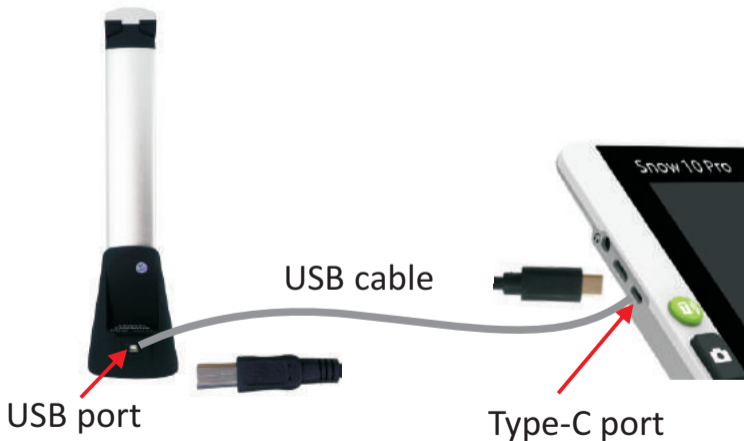

A few seconds later, the Snow 10 Pro will activate the text-to-speech function automatically, and you will hear the voice prompt “text-to-speech function”.

NOTE:

During use, the unit may go back to near-view mode. To use the easeScan, you can unplug the type-c port and plug again.

(2) Align marking of document mat:

- Place the soft document mat on the desktop.
- Place the easeScan alongside the white triangle of document mat. Make sure the triangle on the easeScan aim right at the triangle on the document mat. Please refer to the picture next page.



(3) Start text-to-speech function

- Place the document in the A4 frame of the document mat.
- Press the Freeze button on Snow 10 Pro to capture the document and process it into speech.

NOTE:

To experience the text-to-speech function, please refer to chapter 6 in Snow 10 Pro User's Guide.

4. Specifications

- Weight: 537g (18.9 Oz)
- Packing size (W/H/D): 320 * 82 * 82 mm
(12.6" * 3.2" * 3.2")
- Camera: 5M pixel

ZOOMAX TECHNOLOGY CO., LIMITED

Add.: 9F, Building D, Paradise Software Park,
No.3 Xidoumen Road, Xihu District, Hangzhou,
Zhejiang Province, 310012, China.

Tel: +86-571-87006308

Fax: +86-571-87397220

Email: sales@zoomax.com

Visit us at: www.zoomax.com



Zoom Your Vision, Max Your Life